

G-Premio BOND from GC

One-component light-cured universal adhesive

Selecting the appropriate bonding for a given indication, and making sure to follow precisely the different procedure steps is not always easy. This is why GC developed **G-Premio BOND** - a one-bottle **universal bonding** compatible with **all etching modes** and which can be used not only for **direct bonding**, but also for **repair cases** & **hypersensitivity treatment**.

Moreover, GC wants to offer this ease of use & versatility without any impact on the quality of the bond. G-Premio BOND offers the **advantages of a universal** but with a **top performance in all situations**.



Because we believe in zero compromises.

Why Universal?



Universal in etching modes

The choice is yours!



Self-etch



Selective etch



Total-etch

Universal in indications Multi-use!



Direct bonding*



Repair cases*



Hypersensitivity treatment**





Thanks to a combination of three functional monomers:

Universal in performance

Optimal performance on different substrates!



for a reliable adhesion to



for an excellent adhesion to enamel & dentin, zirconia, alumina and non-precious metals



to bond to precious metals***

^{*}Pictures courtesy of Dr S. Koide, Japan

^{**}Picture courtesy of Dr E. Sauji, Japan

^{***}MDTP: Methacryloyloxydecyl dihydrogen thiophosphate

Believe in zero co

Zero compromises on performance

Strive for zero debondings!

G-Premio BOND: 3μι

Dentin

Displaying an excellent bond strength on both enamel and dentin regardless of the etching mode, G-Premio BOND guarantees an optimal performance and durability in time. Its unique composition based on three functional monomers (4-MET, MDP and MDTP) enables a stable bond not only to tooth structure, but also to composites, metals (including precious), zirconia and alumina.

When combined with Ceramic Primer II, it will create a durable adhesion to any type of

glass ceramics, even after thermocycling. Because we believe that a separate silane will provide the best long-term results.

Take a step towards no failures!

Aim for zero discolorations!

Thanks to an optimal wettability and penetration and a very efficient drying procedure (5 seconds at MAXIMUM air pressure), G-Premio BOND achieves an extremely thin film thickness of 3µm. Although the bonding is very fluid, it contains a high amount of fillers and photo-initiators which help to obtain a strong bonding layer. These characteristics coupled with the HEMA-free formulation guarantee the preservation of the bonding layer in time and prevent any hydrolysis leading to discolorations.

Choose durable aesthetic results!

Seek zero post-operative sensitivity!

With G-Premio BOND, there is no need to worry about open tubules possibly leading to post-operative sensitivity. When using the self-etch and selective etch approaches, the tubules are opened only slightly. In total-etch mode, the tubules will be fully infiltrated thanks to the bonding's exceptional wettability and penetration ability.

Eliminate patient discomfort!

GCC R&D

ompromises


Zero compromises on manipulation

Attempt zero mistakes!

With G-Premio BOND, handling errors are virtually banned from the restorative routine. Combining an easy procedure with a low technique sensitivity, G-Premio BOND offers a complete peace of mind during the procedure:



- The drying step is **impossible to miss**: 5 seconds, MAX air pressure
 - Light-curing is very efficient thanks to a high photoinitiator content
 - Visual control of the application is made easy by the yellowish colour of the bonding before light-curing

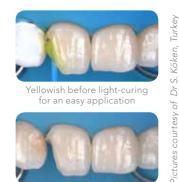

ellowish before light-curing for an easy application



Completely invisible after light-curing for the best aesthetic result







With a working time of 7 minutes in its dedicated dish, G-Premio BOND allows **stress-free working**. It also offers you a versatile solution in one bottle: for direct bonding, but also for repair cases and **hypersensitivity treatment**. The application is made very easy thanks to its **optimal viscosity and wettability**: you could say G-Premio BOND "behaves like water".

Forget about cumbersome routines!

Go after zero waste!

With G-Premio BOND, you can **save precious time** thanks to a very fast procedure: 25s in self-etch, 50s in selective or total-etch. This will be particularly beneficial in situations where isolation is not optimal, or in pediatric cases. G-Premio BOND also helps you to minimise material waste thanks to a unique silicon bottle cover optimising dispensing control and a thin nozzle delivering very small drops - up to 300 per bottle. Stable until the last drop...

Stop wasting time and material!

Zero compromises on Direct Restorations



Select your preferred technique. Etch for 10-15s, rinse & dry

G-Premio BOND displays the highest bond strength values among the products tested after thermocycling, on both enamel and dentin in self-etch mode.



Apply G-Premio BOND and wait for 10s




Dry for 5s at MAX air pressure



Light-cure for 10s

Shear bond Strength (MPa) to enamel and dentin in self-etch mode, after thermocycling (20.000 cycles)

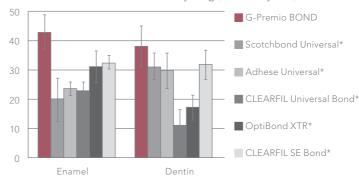

Source: GCC R&D data, Japan, 2015 * Not trademarks of GC

Zero compromises on Repair Cases




G-Premio BOND displays an excellent adhesion to all substrates after thermocycling, including precious metals thanks to the MDTP monomer. The other products tested have a low performance on precious metals.

Primer II) together with G-Premio BOND on glass ceramics enables to obtain a durable adhesion to any type of ceramics.

The use of a separate primer (such as **Ceramic**

Shear bond Strength (MPa) to indirect substrates after thermocycling (5.000 cycles)

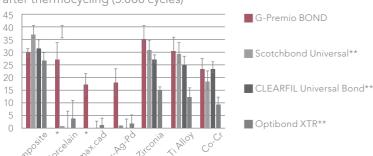

Source: GCC R&D data, Japan, 2015

- * Used in combination with Ceramic Primer II ** Not trademarks of GC

Roughen the surface; Rinse & dry



If there is a glass ceramic surface, apply Ceramic Primer II & drv



ylaaA G-Premio BOND to all surfaces to be repaired and leave for 10s



Dry for 5s at **MAX** air pressure



Light-cure for 10s



G-Premio BOND from GC

Dare to go for...



Zero debondings



Zero mistakes



Zero discolorations



Zero hassle



Zero post-op sensitivity



Zero waste



Article number	Description
009030	G-Premio BOND 5mL Kit
009036	G-Premio BOND 5mL Bottle Refill
009034	G-Premio BOND Unit Dose 50 pcs

GC EUROPE N.V.

Head Office Researchpark Haasrode-Leuven 1240 Interleuvenlaan 33 B-3001 Leuven

Tel. +32.16.74.10.00 Fax. +32.16.40.48.32 info@gceurope.com http://www.gceurope.com

GC UNITED KINGDOM Ltd.

12-15, Coopers Court Newport Pagnell UK-Bucks. MK16 8JS Tel. +44.1908.218.999 Fax. +44.1908.218.900 info@uk.gceurope.com http://uk.gceurope.com

